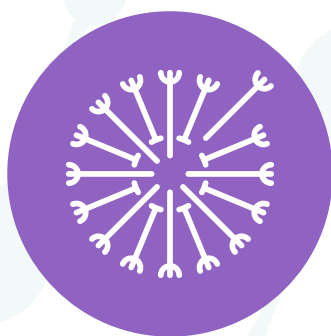
LABORTEST

Allergie Check

Standard Kit

Home Sampling Allergie Check

Im Set enthaltene Artikel / Items Included in the Kit / Articles contenus dans le kit

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Referenz BD Technical Information Sheet, BD Microtainer® Z Tubes REF
365964, TIS365964, CF, 23 05 14, 01
BD Technical Information Sheet, BD Microtainer® Contact-Activated Lancet, WEU270.1

Notes on storage, precautions and expiry of the product

The capillary blood collection set should be stored at room temperature (up to 25°C) in its original packaging until use. Do not heat or freeze the product. Do not use after expiration date, which is clearly printed on each Microtainer tube.

Do not use the product if there is evidence of damage or contamination (both the Microtainer tube and the safety lancets).

After sample collection, the sample should be dropped off at a post office the same day the sample is taken. If this is not possible, the sample can be stored protected from sunlight at room temperature (up to 25°C), for a maximum of 24 hours. Samples that do not arrive at the laboratory within 2 days of collection will not be analysed.

Referenz BD Technical Information Sheet, BD Microtainer® Z Tubes REF
365964, TIS365964, CF, 23 05 14, 01
BD Technical Information Sheet, BD Microtainer® Contact-Activated Lancet, WEU270.1

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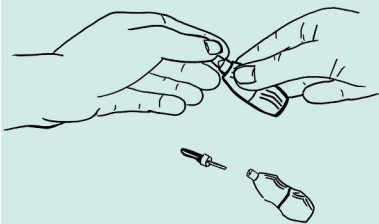
Instructions for capillary blood sample collection

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Wash hands thoroughly with soap and dry well.

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Open the Microtainer tube and place it upright on a flat surface.

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
Disinfect the desired puncture site with the alcohol swab.

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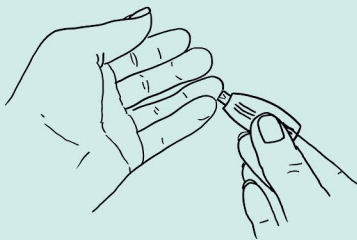
Twist off the safety tab and position the lancet against the desired puncture site. The lancet is activated by pressing firmly against the puncture site, after which the tip retracts automatically into the lancet.

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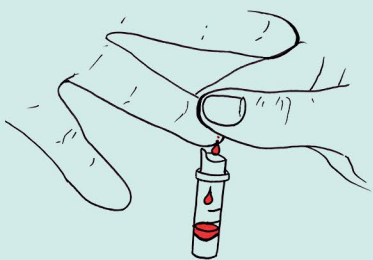Wipe the first drop of blood away with the alcohol swab and collect the emerging blood in the Microtainer tube.

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
Fill the tube to the second fill line. If the minimum amount of 250 µl is not reached, puncture a second finger with the second safety lancet.

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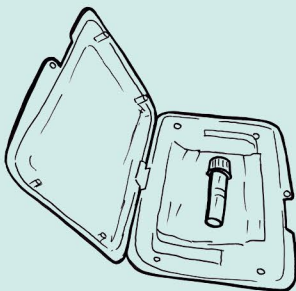If there is enough blood in the Microtainer, close it with the cap and gently mix by inverting the tube 7 times. DO NOT SHAKE!

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Write the name on the enclosed label and stick it on the Microtainer container. Put the tube into the security transport case and seal it. Dispose of the used safety lancets with regular trash using the disposal bag.

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The collection should be carried out on Mondays-Thursdays and should be dropped off at a post office the same day the sample is taken.

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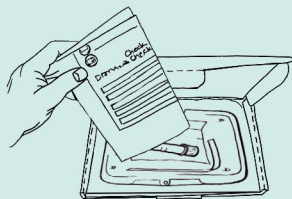
Affix the self-adhesive, pre-stamped return label to the top of the box.


2. Das geschlossene Sicherheitstransport-Etui mit der beschrifteten Probe darin in die Schachtel legen.
Place the closed security transport case (which contains the labelled tube) into the cardboard box.

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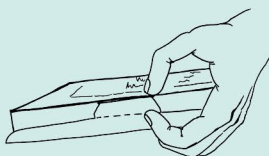In order to guarantee the best possible sample quality, the sample should be dropped off at a post office the same day the sample is taken.