LABORTEST

Ermüdungs Check

Standard Kit

Home Sampling Ermüdungs-Check

Im Set enthaltene Artikel / Items Included in the Kit / Articles contenus dans le kit

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Referenz BD Technical Information Sheet, BD Microtainer® Z Tubes REF
365964, TIS365964, CF, 23 05 14, 01
BD Technical Information Sheet, BD Microtainer® Contact-Activated Lancet, WEU270.1

Notes on storage, precautions and expiry of the product

The capillary blood collection set should be stored at room temperature (up to 25°C) in its original packaging until use. Do not heat or freeze the product. Do not use after expiration date, which is clearly printed on each Microtainer tube.

Do not use the product if there is evidence of damage or contamination (both the Microtainer tube and the safety lancets).

After sample collection, the sample should be dropped off at a post office the same day the sample is taken. If this is not possible, the sample can be stored protected from sunlight at room temperature (up to 25°C), for a maximum of 24 hours. Samples that do not arrive at the laboratory within 2 days of collection will not be analysed.

Referenz BD Technical Information Sheet, BD Microtainer® Z Tubes REF
365964, TIS365964, CF, 23 05 14, 01
BD Technical Information Sheet, BD Microtainer® Contact-Activated Lancet, WEU270.1

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Instructions for capillary blood sample collection

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